


LEM

SELF SUCTION


The SelfSuction LEM allows elastic ligatures aspiration in complete autonomy.

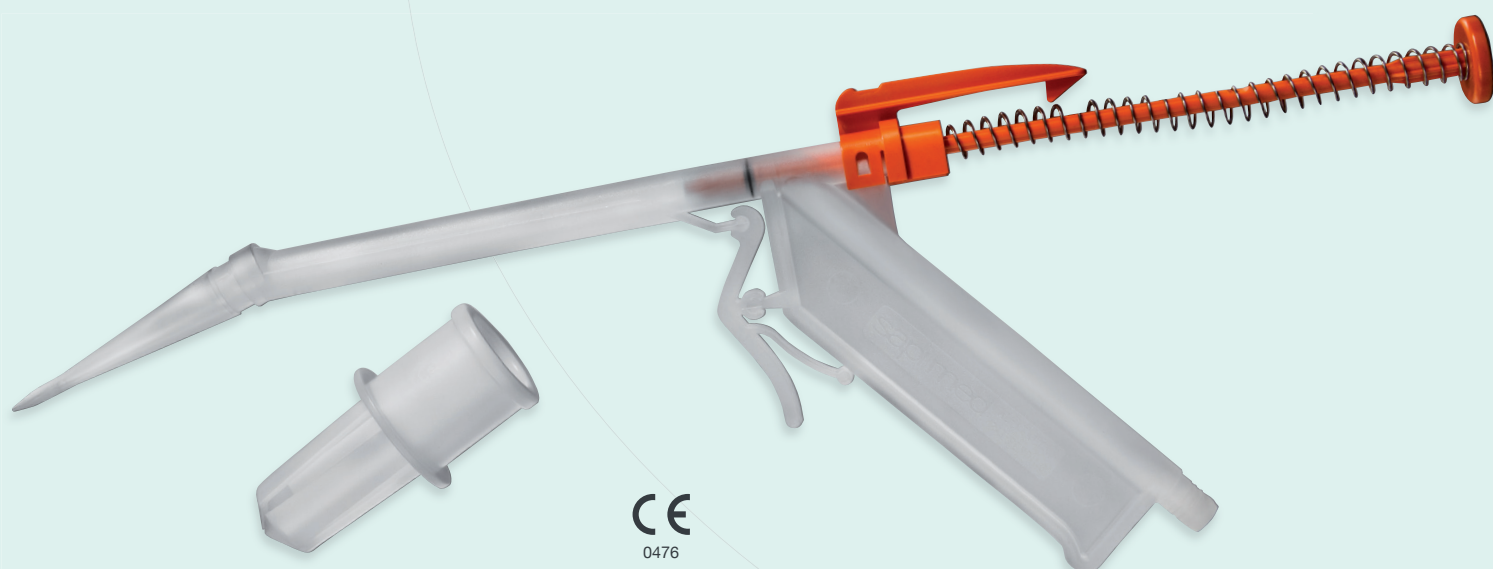

CE
0476

*The SelfSuction LEM is a single-patient device for suction band ligation of haemorrhoids.
No need for an external surgical aspirator.*

Excellent overview of the treatment area.

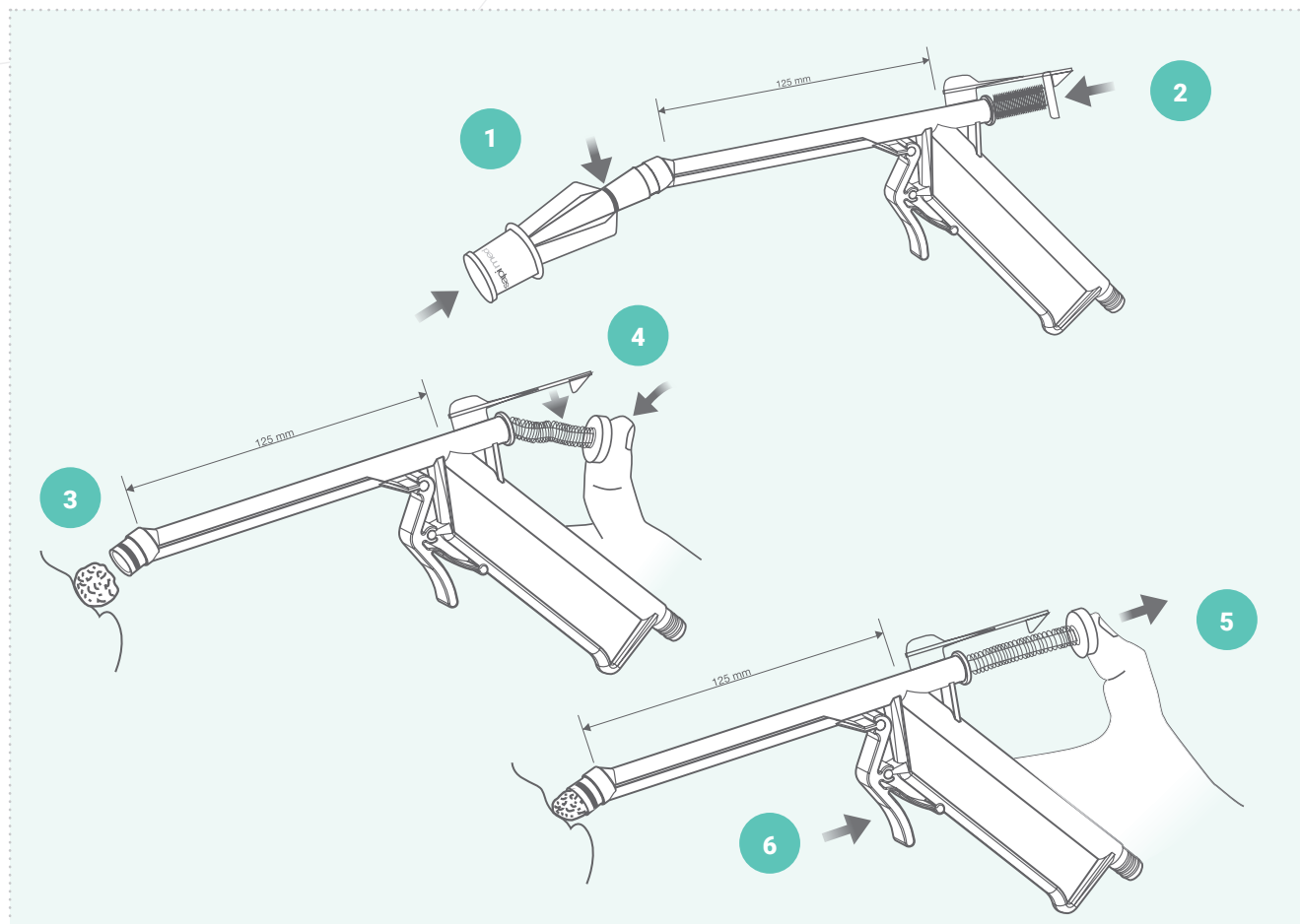

INSTRUCTIONS FOR USE

1. Load the ring onto the device, using the loading mechanism supplied.
2. Press the plunger down until it catches the hook on the device.
3. Push the end of the banding ligator onto the haemorrhoid.
4. Unhook and push down the plunger from the stop.
5. Control the suction by using your thumb to adjust the plunger and waiting for about 30 seconds, until the haemorrhoid is retracted.
6. Pull the trigger to ligate the haemorrhoid with the banding ring.

To extract the LEM, press the plunger to decrease the pressure and push the trigger.

The SelfSuction LEM allows elastic ligatures aspiration in complete autonomy.

- ! The suction capacity of the SelfSuction LEM plunger is -37.000 Pa.
- Do not connect to a suction device.

Also available the following versions:

- **REF A.5670.3:** complete with 3 rings
- **REF A.5670.5:** complete with 5 rings

Non sterile