

A.5680


A.5680LF

LEM4RINGS

sapi med

SURGICAL
LINE

Disposable haemorrhoid ligator for suction and banding with 4 preloaded rings.

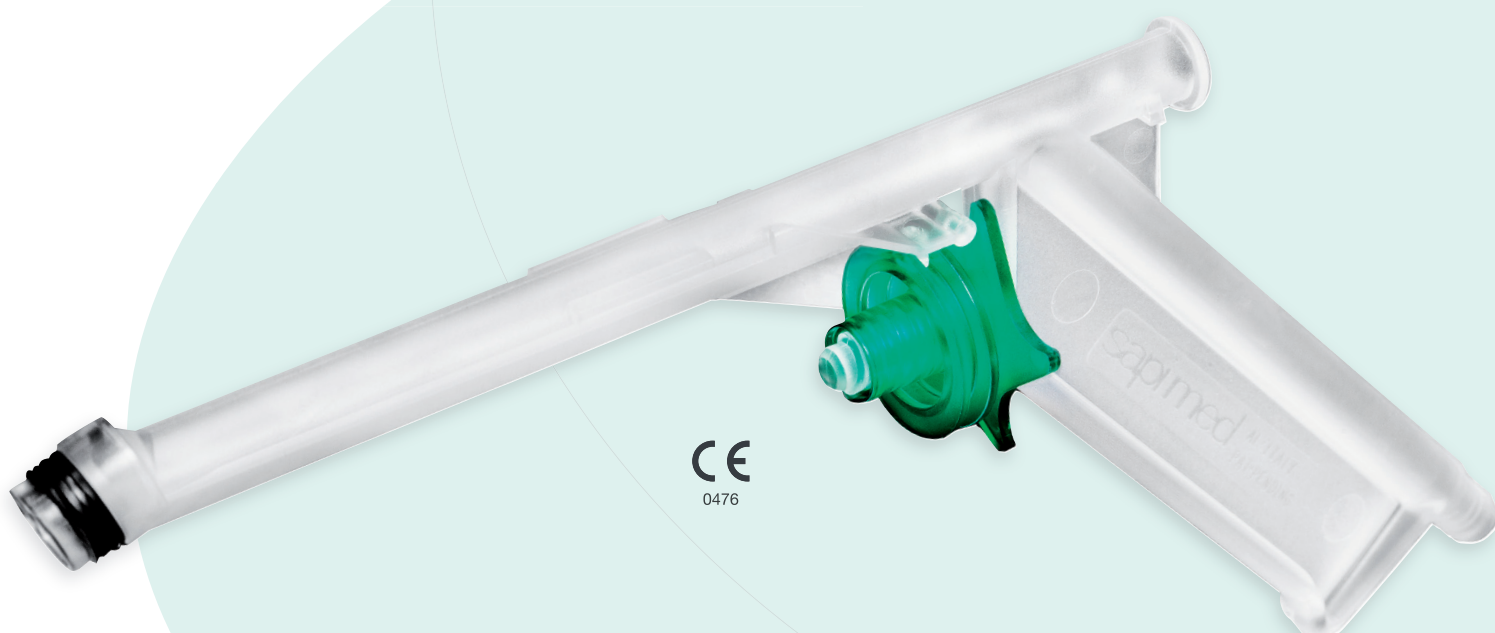

Lem4Rings, the last born of Sapi Med products, completes the extensive range of disposable haemorrhoid ligature devices.

Quick and easy to use, Lem4Rings has 4 preloaded rings that allow the performance multiple elastic ligatures during a single outpatient visit.



INSTRUCTIONS FOR USE

1. Connect Lem4Rings to a low pressure surgical suction device (5-10mm H₂O, 50-100 Pa).
2. Place the elastic ligature on the haemorrhoidal nodule to be tied.
3. Gently suction the lump.
4. Slowly rotate the toothed ring to make the first ring drop to the base of the nodule.
5. The procedure can be repeated to perform up to 4 ligatures, if necessary.

Available in the versions:

REF A.5680 with latex rings

REF A.5680LF with latex-free rings

Not sterile