Dilastor

sapi med

Linea Farmaceutica • Home Care Products

Linea Farmaceutica • Home Care Products

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DILAGENT®

REF A.1004

curative "exercises" for anal fissures, haemorrhoids, hypertonic muscles and postsurgical stenosis

Altered perineal muscle function entails the onset of potentially disabling, painful anorectal diseases. Dilagent offers effective and targeted training for anorectal muscles, encouraging their functional recovery.

the package contains:

- a biomedical silicone probe with pressure valve and 60 cm infusion tube
 - one 60 mL syringe
 - sample of Dilatan cream
 - sample of Sapigen K detergent for personal hygiene
 - directions for use

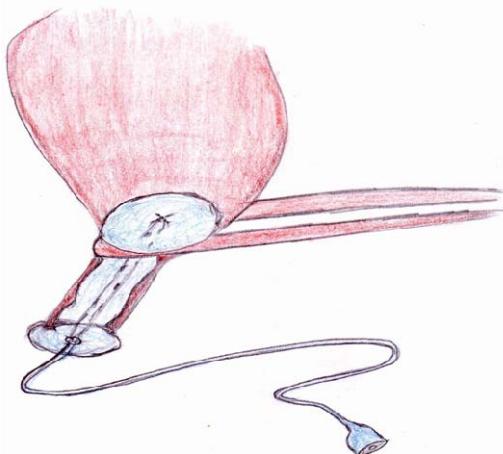
DILAGENT is a **soft silicone** anal dilator. It is indicated for the treatment of anorectal diseases caused by a hypertonic sphincter, namely anal fissures, haemorrhoids and painful spasms after surgical treatment of the anorectal segment. It is also effectively used in cases of postsurgical stenosis of the anal canal.

A hip bath with warm water and **Sapigen K** for 5-8 minutes is recommended before treatment to relax perianal muscles. After using the provided syringe to remove any air contained in the probe, generously lubricate the dilator with **Dilatan Cream** and gently insert it into the anus until the handle comes in contact with the anal rim.

For the first 2 or 3 days, inject 10-20 mL of hot water into the probe (40°/45°). As treatment proceeds, gradually increase the quantity of warm water **to a maximum amount of 50 mL**. Larger amounts might cause a continuous bothersome urge to defecate. The length of the tube enables easy performance of said manoeuvres autonomously.

Anal fissures and, generally, all painful anal conditions (**symptomatic haemorrhoids with muscle hypertonia, haemorrhoidal thromboses, pain after surgery and anal spasms**) are usually associated with a hypertonic internal anal sphincter. The "mushroom-like" shape of the probe facilitates anchoring of the dilator over the puborectalis muscle, transferring heat not only to the internal sphincter but also to the puborectalis muscle itself and to the distal rectal muscles. This important function of Dilagent relaxes the muscles promoting healing for the above mentioned disorders.

Moreover, contraction of the puborectalis muscle (the puborectalis sling) is considered as one of the principal mechanisms involved in maintaining faecal continence. Anal sphincters and the puborectalis sling are released during defecation to allow the anus, anal canal and rectum to straighten, thus facilitating evacuation. If, instead, there is **paradoxical contraction of the puborectalis muscle** during defecation, the enhanced tension of the muscle makes the anorectal angle more acute, thus preventing evacuation of the rectal ampulla (anismus).

The **Dilagent** probe encourages release of the puborectalis muscle through direct transmission of heat and therefore effectively treats anismus.

Treatment must be administered for 3-4 weeks.

After each use, wash the probe with neutral soap and water. Dry with a soft cloth before storage. **The use of this device is strictly personal.**

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Paraben free.

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Intimate Cleanser

Sapigen K intimate cleanser from Sapi Med has been formulated to deal with problems in gynaecology and proctology.

Its surfactants are highly compatible with the skin, and allow delicate yet powerful cleansing of the mucous membranes with good foaming power. The product has strong antibacterial and antimycotics action, and is highly eudermic and highly tolerable. The pH is 5.5, which is compatible with the physiological parameters of the intimate membranes.

In developing this formula, the company used natural elements such as capers, aloe vera, prickly pear, Irish moss, mallow and horsechestnut, recognised for their soothing, moisturising and redness-reducing action, to help reduce anal and genital itching. The product also includes tea tree and eucalyptus essential oils, which are efficacious in preventing bacterial and candida infections, and possess significant deodorising and eutrophic activity.

The product does not contain Sodium Lauryl Sulphate or genetically modified substances.

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Cream with wild chamomile, klamath weed and calendula

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CAMOCAL is a natural remedy for the treatment of anal tract disorders. It is rich in natural elements, protective, lubricant and decongestive. It exploits the active ingredients of calendula, hypericum, horse chestnut, chamomile, allantoin and propolis.

Prolonged ongoing use of **CAMOCAL** soothes epidermal disorders by reconstructing the protective hydro-lipid mantle. There are no contraindications to its frequent use.

The active actions of medicinal plants that are used for the most common preparations are known to ancient traditional herbal medicine. We shall particularly describe some of them that are contained in **CAMOCAL**.

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Anal dilator

DILATAN®PLUS is a cryo-thermal anal dilator available in 5 different sizes only to be used when advised by a physician. Three boxes are available (18/20 mm, 23/27 mm and 30 mm), offering patients the most suitable size for their particular problem and individual anatomy.

DILATAN®PLUS consists of an inert plastic shell with a large disk at the base to allow the patient to grip the device easily. Each sealed dilator contains a thermal jelly which, when heated or cooled, maintains the dilator's temperature during use.

Thermotherapy (warm) or cryotherapy (cold) dilators may be used effectively to treat differing pathologies. For the following conditions the dilator should be used warm, up to but not exceeding 50 degrees Celsius:

- Stenosis (contraction) of the anus, due to cicatricial or idiopathic origin
- Hypertonia or spasm due to, and sustained by, haemorrhoids, proctalgia or fistulas
- Prevention of post-operative stenosis due to haemorrhoids or fistulas

For the following conditions, the dilator should be cooled to not below 0 degrees Celsius:

- Anal bleeding.

For other conditions, the temperature of the dilator does not matter. Sizes 18 and 20 (suitable for women or children) and size 23 are indicated for the treatment of hypertonic anal sphincter, actinic proctitis - often associated with anal stenosis – and in tight anal stenosis.

In spasm and in the treatment of the chronic anal fissures associated with severe hypertonic anal sphincter, the larger dilators (27 and 30) should be used.

Directions for use:

In CRYOTHERAPY, cool the appropriate dilator in a refrigerator or immerse in ice and water for 10-15 minutes.

In THERMOTHERAPY, immerse the dilator in warm water (no greater than 50°C) for 10-15 minutes.

The **DILATAN®PLUS** works well regardless of whether the lower bowel is full or empty. In case of spasm, sitting in a warm water bidet prior to insertion will help relax the sphincter muscles and assist the therapy.

Prior to insertion, the dilator should be covered in a layer of **CREMA DILATAN®** cream.

Sitting in the bidet or lying on the bed, insert the dilator slowly up to its base. Keep it in position for two to three minutes, and then pull it slowly out. It is advisable not to stand or walk with the dilator in position.

- Following haemorrhoidectomy, dilations should be performed once or twice daily for the first month, reducing to three times a week for the second and third months. This will greatly relieve post-operative discomfort.
- In cases of anal stenosis, twice-daily use of the dilator is advisable. Treatment should be continued for several months.
- For other conditions, **DILATAN®PLUS** should be used twice a day whilst symptoms persist, and the treatment continued for at least a month after the symptoms have ceased.

Cleaning DILATAN®PLUS:

DILATAN®PLUS anal dilators must always be washed after use with warm water and neutral soap. If necessary, leave for several hours in a disinfectant solution.

Warnings:

It is recommended a proctological examination to avoid a different origin of symptoms. Keep out of children's reach. Do not swallow.

DILATAN®PLUS is a single-patient product.

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Relief from anal pain
in many common anorectal diseases
(fissures, haemorrhoids, proctalgia)

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**“Dilation and thermotherapy
may avoid surgery”**

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Anal dilator

Anal dilations, as well as internal and sphincterotomy and anal stretching, are proposed methods for the treatment of anal fissures and haemorrhoids. Both of these conditions recognize varying degrees of anal canal hypertonus as the cause and/or effect of the symptomatology. Even without surgically eliminating the anal fissure or reducing the mass of haemorrhoidal tissue, these methods are able to significantly alleviate discomfort; therefore they must be carefully considered when dealing with symptomatic treatment of these afflictions.

The internal anal sphincter may undergo spastic contraction due to various pathological conditions such as constipation with small and hard stools, chronic laxative consumption, infections, haemorrhoidal congestion, dysentery, proctitis, colitis, anal fissure and sympathetic hypertonus. If this spasm persists the anus becomes incapable of relaxation, thereby impeding evacuation of the faeces as well as aggravating the underlying disorder.

If the spasm allows the passage of a dilator, each successive dilatation will be less painful and will allow healing of the lesion by scar tissue formation. The elasticity and distensibility of the anal canal will be regained with either the disappearance of or the improvement of symptoms. In this manner surgery may be avoided.

The anal dilator **DILATAN®** was studied and developed specifically for this type of ailment. Externally it has a cylindrical shape, which becomes conical towards one end. The surface of the dilator is perfectly smooth. The internal compartment is hermetically sealed and contains a jelly which, depending upon the treatment required, is able to retain hot or cold.

It is known that certain anal pathologies improve after undergoing cold treatment (cryotherapy for haemorrhage and oedema) or hot treatment (thermotherapy for muscle spasm and pain due to various factors). The thermal effect adds to the mechanical one (i.e. the exercising of the sphincter muscles during dilatation).

The **DILATAN®** anal dilator by Sapi Med is made of an inert plastic material conforming to the regulated chemical values of the official pharmacopoeia and to **DDM 93/42/CEE**.

Each package containing three dilators of different dimensions (20, 23 and 27mm) is attractive and patient-convenient, and ideal for easy placement in the refrigerator.

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CREMA DILATAN® is a natural remedy for treatment of fissures and haemorrhoids. It has no known secondary reaction, is non-greasy and has a pleasant smell.

Is an adjuvant in DILATAN® / DILATAN® PLUS cryothermal therapy; it is protective, lubricant and decongestant. It exploits the active and natural elements of calendula, klamath weed, horse chestnut, wild chamomile, allantoin and propolis.

Calendula Officinalis

Plant pharmacological studies have suggested that Calendula extracts have anti-viral, anti-genotoxic and anti-inflammatory properties. Calendula cream is used topically to treat acne, reducing inflammation, controlling bleeding and soothing irritated tissue. It is effective in treating radiation dermatitis.

Hypericum Perforatum

St John's wort is widely known as a herbal treatment for mild depression. Hyperforin has also been found to have antibacterial and healing properties against anal and vulvar fissures, wounds and haemorrhoids.

Matricaria Chamomilla

Chamomile, besides being a well-known natural calming, is used medicinally to treat sore stomach, irritable bowel syndrome. Chamomile is used as a mild laxative and is anti-inflammatory and bactericidal.

Haesculus Hippocastanum

Aescin is a mixture of saponins with anti-inflammatory, vasoconstrictor and vasoprotective effects found in aesculus hippocastanum (the horse chestnut). The saponin aescin, however, has been used for health purposes such as varicose veins, edema, sprains.

Allantoin

Allantoin is a chemical non-toxic compound promoting cell proliferation and wound healing; it has a soothing, anti-irritant and skin protectant effect by forming complexes with irritant and sensitizing agents.

Propolis

Natural medicine practitioners use propolis for inflammations, viral diseases, ulcers, superficial burns or scalds. Propolis is considered a natural antibiotic with antifungal properties and also exhibits emollient effects.

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The DILASTOM® stomal dilator is a medical device specifically designed so that patients may carry out home treatment for cutaneous and fascial colostomy stenosis with complete safety and convenience. Surgical operations to correct colostomy stenosis may be avoided by protracted use of DILASTOM® for long periods, without producing any undesirable side effects.

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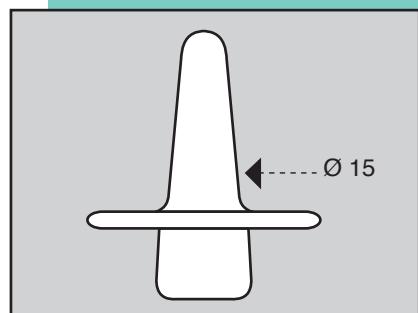
Many cases colostomy is trouble-free for the patient. However, complications can occur. Between 8 and 10% of cases (collated global data) result in stenosis or stricture of the stoma lumen. To accurately define whether or not stenosis has occurred, consideration must be given to the diameter of the stoma, and the elasticity of the intestinal wall and of the surrounding tissue. Colostomy stenoses are described as either precocious or belated (early or late) depending on the post-operative time interval before the stenosis becomes evident. A further categorisation into either cutaneous or fascial has been identified, dependant on the tissue area affected: the muco-cutaneous junction or apical respectively. When stricture occurs, difficulties can be experienced both in the discharge of faeces and in execution of irrigation procedures. The instigation of dilator therapy can stop the progressive stricture of the external colostomy orifice and, if correctly applied, reverse the process.

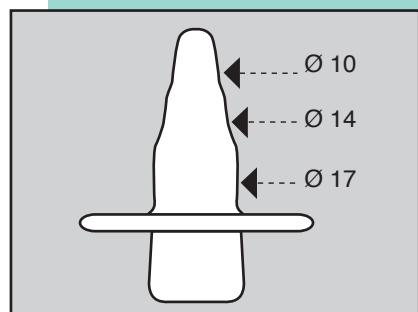
Dilation therapy should begin with the smaller diameter DILASTOM®. Lubricate well with petroleum jelly or other suitable lubricant. Rotate the DILASTOM® slightly and use gentle pressure when inserting into the strictured stoma. The DILASTOM® should be kept in position for 10-15 minutes. Repeat the operation at least every 12 hours. The treatment period is normally between one and two months, moderated according to practitioner advice. The second phase utilises the larger diameter DILASTOM®. Application procedure and treatment period is similar to the smaller-sized device. The unique design of these dilators allows them to stay in position with gentle finger pressure on the base.

There are no recorded cases of adverse side effects from prolonged use of DILASTOM®. Conversely, strong evidence exists in support of the claim that regular use avoids the necessity of stricture-corrective surgical intervention.

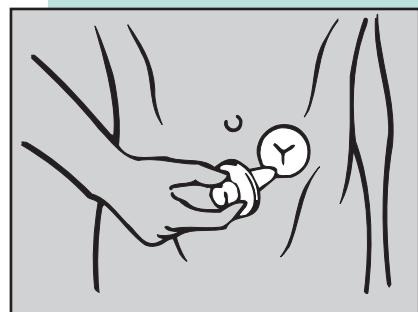
DILASTOM® is supplied in an attractive and patient-convenient small box containing one each of the two sizes. Latex free.

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LUJ from Sapi Med is transparent, water-soluble jelly that provides continuous and prolonged lubrication. It is compatible with the mucous membranes, flexible endoscopic instruments and surgical instruments. The line offers a wide range of packages and formats to satisfy the needs of both specialist and patient. **Paraben free.**

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500ml dispenser in recyclable plastic. **Non sterile** jelly for rectal and gynaecological examinations, and to lubricate flexible endoscopic instruments.

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250ml tube. **Non sterile** jelly for rectal and gynaecological examinations, and to lubricate flexible endoscopic instruments.

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10ml single use sachet **sterile** jelly for catheter insertion, rectal and gynaecological examinations, and to lubricate flexible endoscopic instruments.

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Package: Box of 20 sachets.

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10ml single use sachet **sterile** lubricating jelly for catheter insertion, rectal and gynaecological examinations, and to lubricate flexible endoscopic instruments.

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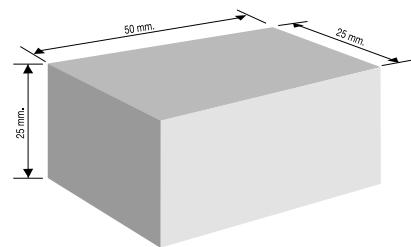
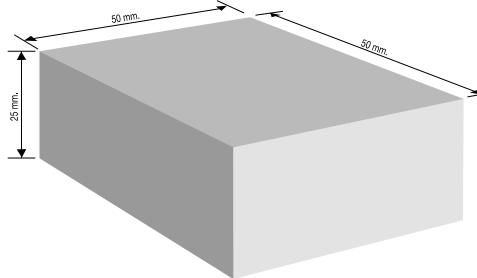
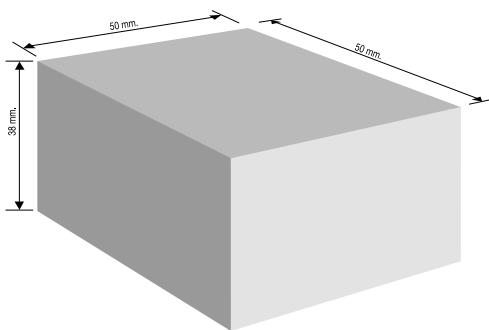
Managing wounds of various surface areas and depths, consequent on surgical operations, is onerous for both the patient and the healthcare system. Frequent medication is necessary for long periods of time, involving a considerable investment of time by medical and paramedical staff.

The use of a disposable, sterile, biocompatible sponge in polyurethane wadding with a detergent effect constitutes substantial progress in medicating systems compared with gauze packing.

Applied inside the wound, it is soaked with exudates after a few hours. Changing Spongy every 6-8 hours allows for frequent evacuation of serum and fibrous or purulent secretions, which will facilitate the granulation of the repair tissue and thus promote healing. It is indicated for use after perfect haemostasis has been achieved by packing the wounds in the operating theatre, replaced it on the first and, if necessary, on the second day after the operation.

Spongy allows the patient to medicate him or herself quickly, and in a familiar environment.

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Instructions for use

The first time the physician should show the patient how to medicate his or her wound with **Spongy**.

Trim Spongy into a shape as close as possible to that of the wound with scissors that have been thoroughly disinfected with hydrogen peroxide, **moisten with physiological solution, squeeze dry and place inside the wound**, applying light pressure to ensure the edges of the sponge touch the walls of the wound. A gauze taped to the skin will keep the sponge in the correct position until it is to be changed, 6-8 hours later, or as advised by the physician. In subsequent cleansing operations the patient can shape the other sponges in a similar way – the sizes will gradually reduce – until the wound is completely healed. Every 7/15 days physician can evaluate whether or not the medication is correctly performed.

Using **Spongy** instead of traditional medication with packing offers the following advantages:

- the sponge may be changed several times a day, with better wound cleansing;
- patients may medicate the wound themselves, with a considerable advantage in organisational terms;
- frequent wound cleansing promotes faster scar formation and healing.

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REF A.6210

CE
0546

Radiopaque markers to study intestinal transit times. For use with intestinal imaging

Time Markers are made up of semispherical grains (3.5 x 3.5 mm) including 75% of polypropylene (food grade). 25% of barium sulphate is added to obtain a specific weight higher than 1. Markers are used to determine intestinal transit times in patients for whom a change of diet produced no improvement in symptoms of constipation, where diseases of expulsion may be suspected, or where a degree of colic constipation with slowing down of total transit times is manifest. The main intended use of Time-Markers is to follow the precise path of food through the entire GI tract, and specifically in the bowel, whilst not influencing the performance of normal gastric and intestinal function.

Latex free.

REF A.6200

Sachets of 30 cylindrical markers:

- Administer an entire sachet of 30 cylinders with food (yoghurt, purée, soup).
- Perform 3 radiographic images on successive days.

Quantity per box: 20x sachets

REF A.6210

Sachets of 10 cylindrical markers:

- Administer a bag of 10 markers daily over 10 days
- Perform 1 radiograph on the 11th day.

Quantity per box: 20x sachets

sapi med
s.p.a.

Sapi Med spa

Via della Chimica, 6 / Zona Industriale D3
15121 Alessandria · Italy
Tel. +39 0131 348109 · Fax +39 0131 348383
www.sapimed.com · info@sapimed.com